

INSTRUCTION SHEET

ElectroLast™ AED Trainer Electrode Peel-Off Pads (POEP)

CPR
P R O M P T®

Installing



Remove manufacturer's electrode pad liner



Stick manufacturer's electrode to ElectroLast™ pad



Remove ElectroLast™ liner



Place on Manikin

Replacing



Place sticky side of ElectroLast™ Peel Off Pad onto trainer electrode pad liner.



On a flat surface, firmly hold down the ElectroLast™ Peel Off Pad and pull up and back the trainer electrode pad.



Continue to hold ElectroLast™ pad while removing the trainer electrode pad. Discard ElectroLast™ pad and replace.