

Instruction Guide for Brad and JT BRAD CPR™ Manikin



Thank you for buying an American made product.

This instruction guide will familiarize you with Simulaids' BRAD CPR™ and JT Brad CPR™ Manikins. Designed with your training needs in mind, BRAD and JT Brad CPR's realistic looks, accurate anatomical landmarks and individual-use mouth/nose pieces with disposable airway/lung system and non-rebreathing valve will continue to reinforce the manikins value. The Manikin's sturdy vinyl covered torso means quick, wipe-clean maintenance.

The below photo shows the jaw thrust maneuver featured only with JT Brad manikins. Use of a water-based lubricant between the jaw and the head will assist in the ease operating this feature.



Brad / JT Brad airway Removal

1. Remove mouth/nose piece from the manikin's face cavity.
2. Remove the connector from the one-way valve stem from inside the face cavity.
3. Lay the manikin on its back and pull the chest skin overlay off the shoulders and down toward the stomach below the chest plate and lung.
4. Tilt the head back and reach under the chin and pull the one-way valve out of the hole in the head.
5. Slide the bottom lung slots off from the chest plate and remove the lung from behind the chest plate and discard the lung.

Brad / JT Brad Airway Installation

1. Select a plastic exhaust manifold cap and install it on the opened port of the one-way next to the lung bag.
2. Lay the manikin on its back. Tilt the head back and grasp the airway assembly by the one-way valve and insert it through the hole in the face by guiding the one-way valve under the chin, with the plastic exhaust manifold cap towards the back of the head (fig. 1). Be sure that the airway tube is forward and lays flat. Press the connector onto the one-way valve stem (fig. 2) **DO NOT PRESS** straight on. Angle and apply a twisting motion while pressing forward
3. Lay the lung behind the chest plate and secure the two slots in the bottom of the lung by placing the slots over the bottom ribs of the chest plate.
4. Pull the lung up until it rests against the neck collar. If the lung is too low, it will detach from the manikin's head.
5. Make sure the chest plate is properly positioned and pull the chest skin overlay back up over the manikin's shoulders.
6. Replace the mouth/nose piece on to the face cavity.

Fig:1


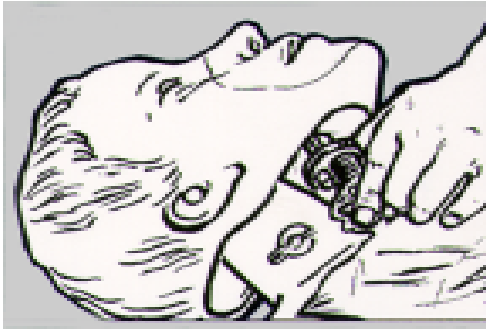


Fig:2

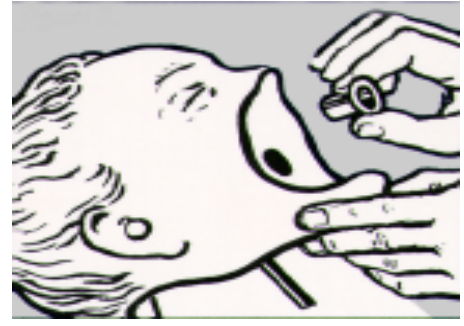

Cleaning & Disinfecting:

Manikin

Wash with warm, soapy water.

Mouth/ Nose pieces

Although Simulaids recommends disposing of Mouth/ Nose pieces at the end of class, they may be reused if desired using the following cleaning instructions.

1. Clean in mild soapy water.
2. Rinse thoroughly.
3. Follow local protocol for disinfection.
4. Rinse thoroughly.

DO NOT USE BLEACH FULL STRENGTH

Part #	Description
2801	Brad with carry bag
2804	JT Brad with carry bag
2805	Airway/ lung system (24/ pk.)
2023	Mouth/ Nose Piece (10/pk.)
2526	Carry bag with kneeling pads

Return Policy

Should it be necessary to return an item for any reason, contact our Customer Service Department to obtain an RGA Number. Please refer to your invoice number when phoning in your request for returning merchandise. Should you have any questions or wish further information on any product we manufacture call our Customer Service Department.

(845)-679-2475

Toll Free: (800)-431-4310 Fax: (845)-679-8996

www.simulaids.com

Simulaids warrants this product to be free from any defect in materials and/or workmanship for a period of three years from the date of purchase, as evidenced by the date of invoice when the product was shipped to the end user. This warranty expressly does not cover abuse, accidental or purposeful damage, or any form of modification to the product. Simulaids reserves the right to either repair or replace affected parts or the entire unit, at their sole discretion, after investigating and reviewing the actual product and the damage. In most instances, a digital photo of the product in question showing the damage will help qualify a product for return to the factory. At no time will any product be accepted at the plant without proper return authorization issued by Simulaids.

Freight and Shipping charges are the sole responsibility of the end user. No product will be received with shipping charges due. Any product considered for warranty work must be identified by serial number and invoice number from the agency through whom the product was purchased. Without this information the product will not receive a return authorization number as required.



Manufacturers of Training Manikins, Casualty
Simulation Kits, Medical Training Devices
(845)-679-2475
Toll Free: (800)-431-4310 Fax: (845)-679-8996
www.simulaids.com