



Economy Sani-Man

Directions for Assembly




Item Numbers 2144 & 2145

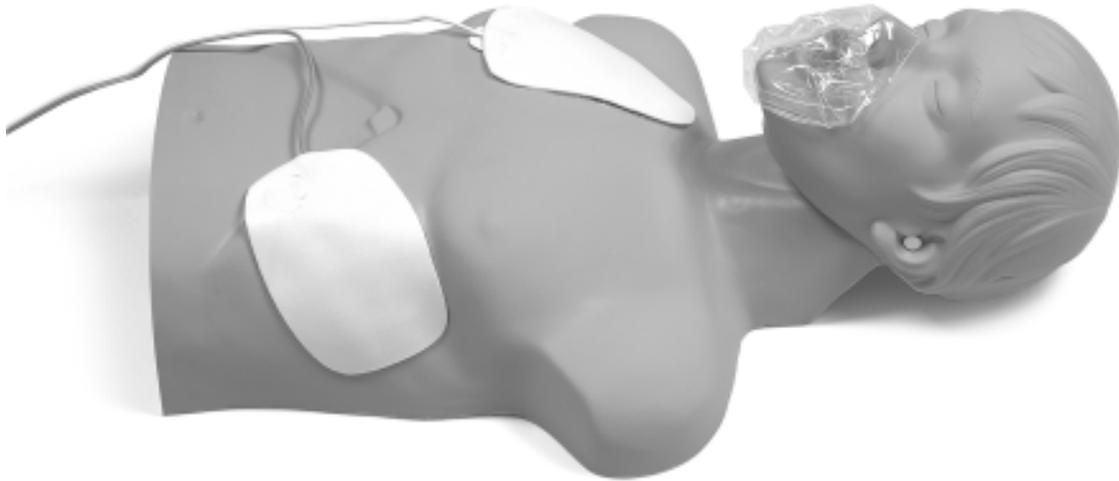


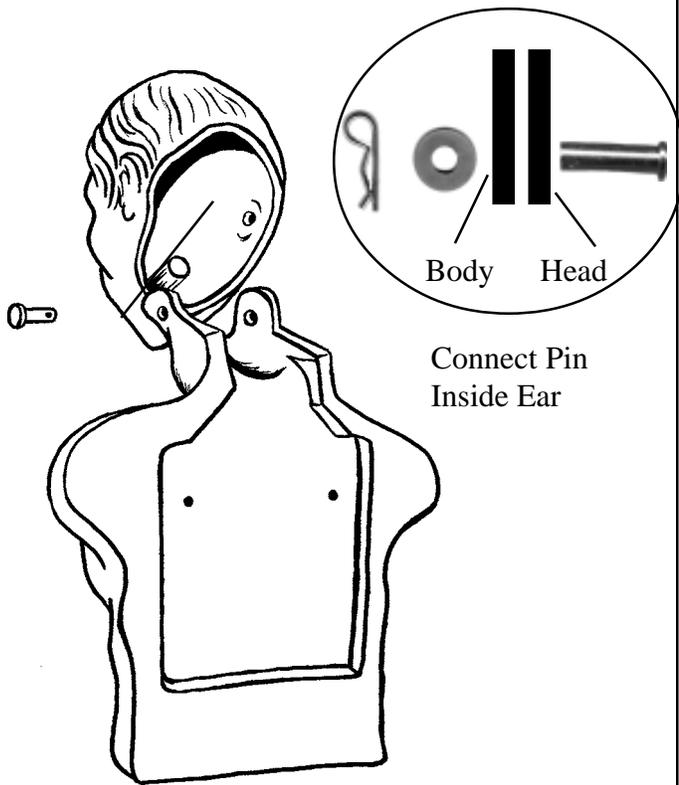



Step 1 Insert the pin into the ear and through the neck then add the washer and connecting pin on the inside of the head.

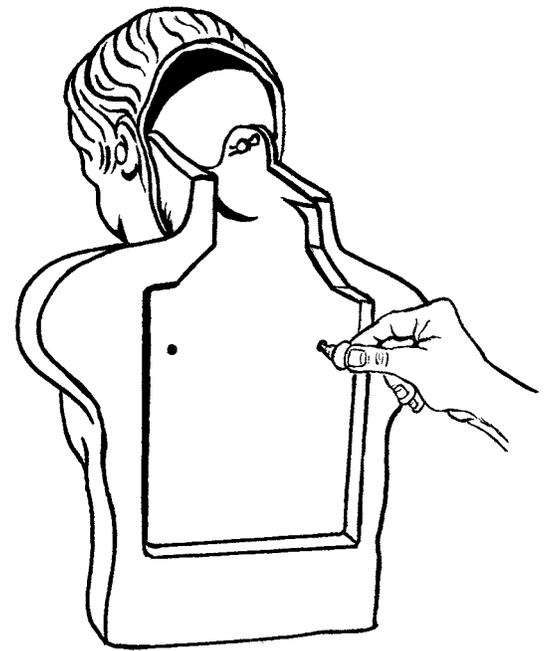

Step 2 Insert the push pins (2) into the back of the manikin.

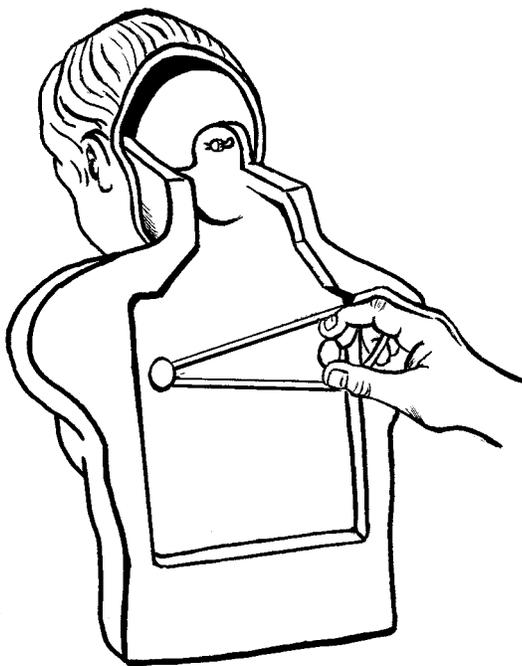


Step 3 Place enclosed rubber band over the push pins.

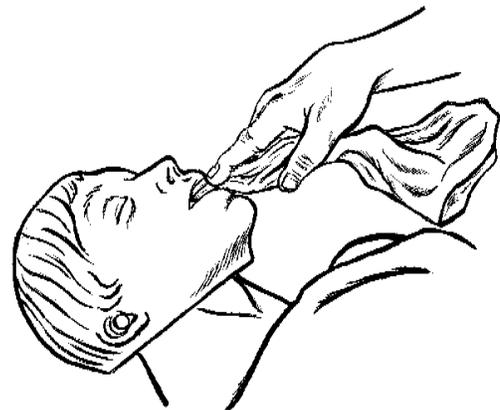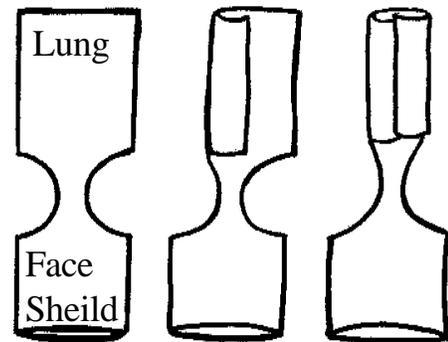

Step 4 Fold the bag lengthwise to facilitate pushing the bag through the mouth. Tilt the head back and place the closed end into the tube in the mouth and push it through to the back.




Step 5 Pull the sealed end of face shield bag through leaving approximately 2” remaining outside of the mouth to allow using it for a face shield.

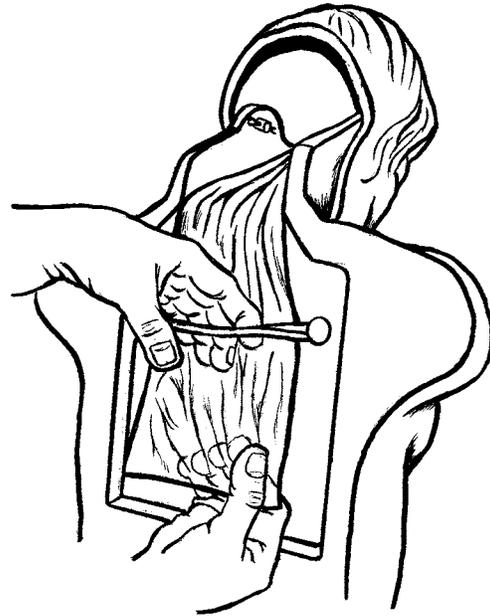

Step 6 Flatten the sealed end of the face shield against the manikin and secure it with the rubber band.



Step 7 Spread open end of face shield bag over nose and chin. Give finger sweep to open bag properly so that there are no twists in the airway.

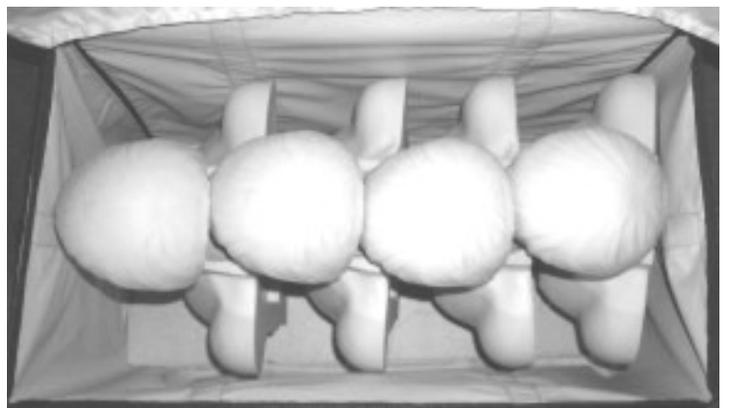

Step 8 Place manikins in carry bag as shown.

Note: The manikin you have just purchased is a member of Simulaids' Sani-Man line. These manikins serve many individuals during a training session. In order to eliminate cross-contamination, or the possibility of cross-contamination, these manikins are designed with a "single use airway." That means that only one person uses the plastic bag airway system.

When the individual completes his/her airway inflations, the bag is to be removed and the next student will place his/her airway in the manikin. The airway can be retained by an individual for further use during that training session, but the airway should be discarded at the end of that session. This is the only recommended procedure for eliminating cross-contamination.

Cleaning : Manikin Wash with warm, soapy water.

AED Pads: Apply Turtle Wax before placing AED pads, wash with warm soapy water after removal.

WARRANTY: Simulaids warrants this product to be free from any defect in materials and/or workmanship for a period of three years from the date of purchase, as evidenced by the date of invoice when the product was shipped to the end user. This warranty expressly does not cover abuse, accidental or purposeful damage, or any form of modification to the product. Simulaids reserves the right to either repair or replace affected parts or the entire unit, at their sole discretion, after investigating and reviewing the actual product and the damage. In most instances, a digital photo of the product in question showing the damage will help qualify a product for return to the factory. At no time will any product be accepted at the plant without proper return authorization issued by Simulaids.

Freight and Shipping charges are the sole responsibility of the end user. No product will be received with shipping charges due. Any product considered for warranty work must be identified by serial number and invoice number from the agency through whom the product was purchased. Without this information the product will not receive a return authorization number as required.

RETURN POLICY: Should it be necessary to return an item for credit for any reason, contact our Customer Service Department to obtain an RGA Number. Please refer to your invoice number when phoning in your request for returning merchandise. Should you have any questions or wish further information on any product we manufacture call or write our Customer Service Department.



PO Box 1289 - 16 Simulaids Drive
Saugerties NY 12477

Toll Free: (800)-431-4310 Fax: (845)-679-8996

www.simulaids.com

4/2006