



---

## RESCUE BABY & CHILD INSTRUCTIONS

---



To simulate realistic weight and to use in water rescue and safety training, fill manikin with water using the following instructions:

1. Hold manikin upside down or lay in sink.
2. Open both valves on bottom of manikin's feet. Rescue Timmy has one metal valve and one aqua drain valve with an internal flap.
3. Attach hose from faucet to left metal valve on baby's foot or insert through internal flap of aqua valve on Timmy's foot.
4. Fill manikin with warm water until water begins to flow from the opened metal valve on right foot.
5. Shut off flow of water.
6. Close valve on right foot.
7. Remove hose from valve on left foot.
8. Gently squeeze air from manikin until water flows from left valve. Close valve tightly.
9. Let manikin lay 1 or 2 minutes with head down.
10. Repeat steps 2 through 8 making sure all air is out of the manikin.

NOTE: Do not store manikins filled with water. Microbiologic growth could occur and extreme temperature will effect the manikin. After every use the manikin must be drained of all water and filled with air before storing it.

1. Place manikins' feet into sink and open both valves. Insert pencil or dowel (DO NOT USE PEN) through internal flap of aqua drain valve for Timmy manikins only.
2. Let drain 30 minutes.
3. Holding manikin over sink squeeze excess water from manikin.
4. Repeat steps 2 and 3.
5. Close metal valve on left foot of baby manikin. Close aqua drain valve cap on Timmy manikin.
6. Attach tubing from inflation bulb to metal valve on right foot. Inflate manikin with air by pumping bulb. DO NOT OVERINFLATE. Grasp valve with tubing and turn to close.
7. Remove inflation bulb assembly.

No. 1350 Rescue Cathy - Newborn Size  
Water filled 7 lbs.  
Sh. Wt. 5 lbs. Size 26" x 8" x 8"

No. 1352 Rescue Billy - 6 to 9 Month old  
Water filled 12 lbs.  
Sh. Wt. 7 lbs. Size 26" x 8" x 8"

No. 2252 Carry Bag for Rescue Cathy and Billy

No. 1351 Rescue Timmy - 3 Year Old  
Water filled 20 lbs.  
Sh. Wt. 10 lbs. Size 34" x 11" x 7"

No. 2260 Carry Bag for Rescue Timmy

## **RETURN POLICY**

Should it be necessary to return an item for credit for any reason, contact our Customer Service Department to obtain an RGA Number. Please refer to your invoice number when phoning in your request for returning merchandise. Should you have any questions, or wish further information on any product we manufacture, call or write our Customer Service Department.

**Warranty:** Simulaids warrants this product to be free from any defect in materials and/or workmanship for a period of three years from the date of purchase, as evidenced by the date of invoice when the product was shipped to the end user. This warranty expressly does not cover abuse, accidental or purposeful damage, or any form of modification to the product. Simulaids reserves the right to either repair or replace affected parts or the entire unit, at their sole discretion, after investigating and reviewing the actual product and the damage. In most instances, a digital photo of the product in question showing the damage will help qualify a product for return to the factory. At no time will any product be accepted at the plant without proper return authorization issued by Simulaids.

Freight and Shipping charges are the sole responsibility of the end user. No product will be received with shipping charges due. Any product considered for warranty work must be identified by serial number and invoice number from the agency through whom the product was purchased. Without this information the product will not receive a return authorization number as required above.

(845)-679-2475

Toll Free: (800)-431-4310 Fax: (845)-679-8996

[www.simulaids.com](http://www.simulaids.com)



 **Simulaids**



Manufacturers of Training Manikins, Casualty Simulation  
Kits, Medical Training Devices  
PO Box 1289-16 Simulaids Dr-Saugerties NY 12477  
(845)-679-2475  
Toll Free: (800)-431-4310 Fax: (845)-679-8996  
[www.simulaids.com](http://www.simulaids.com)