

Nasco
Life/form®

**SURGICAL BANDAGING
SIMULATOR "SURGICAL SALLY"
LF00929U
INSTRUCTION MANUAL**



Life/form® Products by NASCO

LIST OF FEATURES

The simulator features the following simulated wounds:

- Thyroidectomy
- Mid-sternal split – with chest tube drain
- Mastectomy – with simulated drain
- Cholecystectomy – with simulated T-tube
- Laparotomy
- Appendectomy
- Colostomy
- Ileostomy
- Abdominal hysterectomy
- Thoracotomy
- Nephrectomy
- Laminectomy
- Sacral decubitus ulcer – Stage 2
- Leg amputation stump

The **Life/form**® “Surgical Sally” Bandaging Simulator comes complete with a durable carrying case. Sh. wt. 26 lbs.



ABOUT THE SIMULATOR

The **Life/form**® “Surgical Sally” Bandaging Simulator is a replica of a 29” x 17” female with 14 surgical wounds, including a mid-sternal split with chest tube drain, a sacral decubitus ulcer – stage 2, and a leg amputation stump. The skin has also been reformulated so it is drier, allowing bandages to adhere better. Wound closures have been updated with surgical staples for the mid-sternal split, thoracotomy, nephrectomy, laparotomy, abdominal hysterectomy, and amputation stump.

CAUTIONS:

1. Never place the simulator on any kind of printed paper or plastic. These materials will transfer indelible stains. Ball point pens will also make indelible stains.
2. Do not use solvents or corrosive materials to remove residual adhesives. Use only Glue Off (LF01023U) or warranty will is void.
3. Do not use povidone-iodone prep solutions. These solutions will indelibly stain the simulator.

CARE OF SIIMULATOR

Normal soil can be removed with a mild soap and water. REN Cleaner (W9919U) will remove stubborn stains from simulator. Simply apply REN to soiled area and wipe clean with a soft cloth or paper towel.

**SUPPLIES/REPLACEMENT PARTS FOR
THE *Life/form*® “SURGICAL SALLY”
BANDAGING SIMULATOR**

LF01023U	NASCO Glue Off
W09919U	REN Cleaner, 12-oz. bottle

Other Available *Life/form*® Simulators

- LF00698U** Adult Injectable Arm (White)
LF00856U Female Catheterization
LF00901U Prostate Examination
LF00906U Ostomy Care
LF00929U Surgical Bandaging
LF00957U Enema Administration
LF00958U Pediatric Injectable Arm
LF00961U Intramuscular Injection
LF00984U Breast Examination
LF00995U Arterial Puncture Arm
LF00997U Adult Injectable Arm (Black)
LF00999U Pediatric Injectable Head
LF01008U Intradermal Injection Arm
LF01012U Heart Catheterization (TPN)
LF01019U Ear Examination
LF01020U Supplementary Ear Set
LF01025U Male Cath-Ed I
LF01026U Female Cath-Ed II
LF01027U Peritoneal Dialysis
LF01028U Suture Practice Arm
LF01036U Spinal Injection
LF01037U Hemodialysis Arm
LF01062U Pelvic, Normal & Abnormal
LF01063U Stump Bandaging, Upper
LF01064U Stump Bandaging, Lower
LF01069U Cervical Effacement
LF01070U Birthing Station
LF01082U Cricothyrotomy
LF01083U Tracheostomy Care
LF01084U Sigmoidoscopic Examination
LF01087U Central Venous Cannulation
LF01095U Blood Pressure Arm
LF01108U Intraosseous Infusion Simulator
LF01121U Advanced IV Arm
LF01139U Advanced IV Hand
LF01142U Auscultation Trainer
LF01162U Venatech IV Trainer
LF03000U **CPARLENE**® Series
LF03601U Adult Airway Management Trainer
LF03602U Adult Airway Management on Manikin
LF03603U Adult Airway Management Head Only
LF03609U Child Airway Management Trainer
- LF03610U** Child Airway Management Trainer Head Only
LF03611U Child Defibrillation Chest Skin
LF03612U Child IV Arm
LF03613U Child Blood Pressure Arm
LF03614U Child Intraosseous Infusion/Femoral Access Leg Only
LF03615U Complete Child **CRiSis**™ Update Kit
LF03616U Child **CRiSis**™ Manikin
LF03617U Deluxe Child **CRiSis**™ Manikin with Arrhythmia Tutor
LF03620U PALS Update Kit
LF03621U Infant Airway Management Trainer Head Only
LF03622U Intraosseous Infusion Right Leg
LF03623U Infant Airway Management Trainer
LF03626U Child Femoral Access Injection Pad Replacement
LF03632U Child Intraosseous Infusion/Femoral Access Leg on a Stand
LF03633U Child Airway Management Trainer with Torso
LF03693U **Basic Buddy** CPR Manikin
LF03699U "Airway Larry" Airway Management Trainer
LF03720U **Baby Buddy** Infant CPR Manikin
LF03953U **CRiSis**™ Manikin
LF03955U Deluxe **CRiSis**™ Manikin
LF03965U Deluxe "Plus" **CRiSis**™ Manikin
LF04001U **GERi**™ Nursing Manikin
LF04020U **KERi**™ Nursing Manikin
LF04021U **KERi**™ Basic Manikin
LF04022U **KERi**™ Advanced Manikin
LF04030U **GERi**™ Advanced Manikin
LF04040U **GERi**™ Basic Manikin
LF06001U **CPR Prompt**™ Adult/Child Manikin
LF06012U **CPR Prompt**™ Infant Manikin
LF06200U **CPR Prompt**™ Keychain Rescue Aid
LF06204U **CPR Prompt**™ Rescue and Practice Aid

*Nasco*Fort Atkinson

901 Janesville Avenue, P.O. Box 901
Fort Atkinson, Wisconsin 53538-0901
1-800-558-9595

www.eNASCO.com • E-mail: lifeform@eNASCO.com