

Other Available *Life/form*® Simulators

LF00698U Adult Injectable Arm (White)	LF03610U Child Airway Management Trainer Head Only
LF00856U Female Catheterization	LF03611U Child Defibrillation Chest Skin
LF00901U Prostate Examination	LF03612U Child IV Arm
LF00906U Ostomy Care	LF03613U Child Blood Pressure Arm
LF00929U Surgical Bandaging	LF03614U Child Intraosseous Infusion/ Femoral Access Leg Only
LF00957U Enema Administration	LF03615U Complete Child CRISIS ™ Update Kit
LF00958U Pediatric Injectable Arm	LF03616U Child CRISIS ™ Manikin
LF00961U Intramuscular Injection	LF03617U Deluxe Child CRISIS ™ Manikin with Arrhythmia Tutor
LF00984U Breast Examination	LF03620U PALS Update Kit
LF00995U Arterial Puncture Arm	LF03621U Infant Airway Management Trainer Head Only
LF00997U Adult Injectable Arm (Black)	LF03622U Intraosseous Infusion Right Leg
LF00999U Pediatric Injectable Head	LF03623U Infant Airway Management Trainer
LF01008U Intradermal Injection Arm	LF03626U Child Femoral Access Injection Pad Replacement
LF01012U Heart Catheterization (TPN)	LF03632U Child Intraosseous Infusion/ Femoral Access Leg on a Stand
LF01019U Ear Examination	LF03633U Child Airway Management Trainer with Torso
LF01020U Supplementary Ear Set	LF03693U Basic Buddy CPR Manikin
LF01025U Male Cath-Ed I	LF03699U "Airway Larry" Airway Management Trainer
LF01026U Female Cath-Ed II	LF03720U Baby Buddy Infant CPR Manikin
LF01027U Peritoneal Dialysis	LF03953U CRISIS ™ Manikin
LF01028U Suture Practice Arm	LF03955U Deluxe CRISIS ™ Manikin
LF01036U Spinal Injection	LF03965U Deluxe "Plus" CRISIS ™ Manikin
LF01037U Hemodialysis Arm	LF04001U GERI ™ Nursing Manikin
LF01062U Pelvic, Normal & Abnormal	LF04020U KERI ™ Nursing Manikin
LF01063U Stump Bandaging, Upper	LF04021U KERI ™ Basic Manikin
LF01064U Stump Bandaging, Lower	LF04022U KERI ™ Advanced Manikin
LF01069U Cervical Effacement	LF04030U GERI ™ Advanced Manikin
LF01070U Birthing Station	LF04040U GERI ™ Basic Manikin
LF01082U Cricothyrotomy	LF06001U CPR Prompt ™ Adult/Child Manikin
LF01083U Tracheostomy Care	LF06012U CPR Prompt ™ Infant Manikin
LF01084U Sigmoidoscopic Examination	LF06200U CPR Prompt ™ Keychain Rescue Aid
LF01087U Central Venous Cannulation	LF06204U CPR Prompt ™ Rescue and Practice Aid
LF01095U Blood Pressure Arm	
LF01108U Intraosseous Infusion Simulator	
LF01121U Advanced IV Arm	
LF01139U Advanced IV Hand	
LF01142U Auscultation Trainer	
LF01162U Venatech IV Trainer	
LF03000U CPARLENE ® Series	
LF03601U Adult Airway Management Trainer	
LF03602U Adult Airway Management on Manikin	
LF03603U Adult Airway Management Head Only	
LF03609U Child Airway Management Trainer	

Nasco Fort Atkinson

901 Janesville Avenue, P.O. Box 901
Fort Atkinson, Wisconsin 53538-0901
1-800-558-9595

www.eNasco.com • E-mail: lifeform@eNasco.com

COPYRIGHT © NASCO 1983 PRINTED IN U.S.A. NP 269-83/RV 8-05

Nasco
Life/form®

PELVIC EXAMINATION SIMULATOR LF01062U INSTRUCTION MANUAL



Life/form® Products by NASCO

Other Available *Life/form*® Simulators

LF00698U Adult Injectable Arm (White)	LF03610U Child Airway Management Trainer Head Only
LF00856U Female Catheterization	LF03611U Child Defibrillation Chest Skin
LF00901U Prostate Examination	LF03612U Child IV Arm
LF00906U Ostomy Care	LF03613U Child Blood Pressure Arm
LF00929U Surgical Bandaging	LF03614U Child Intraosseous Infusion/ Femoral Access Leg Only
LF00957U Enema Administration	LF03615U Complete Child CRISIS ™ Update Kit
LF00958U Pediatric Injectable Arm	LF03616U Child CRISIS ™ Manikin
LF00961U Intramuscular Injection	LF03617U Deluxe Child CRISIS ™ Manikin with Arrhythmia Tutor
LF00984U Breast Examination	LF03620U PALS Update Kit
LF00995U Arterial Puncture Arm	LF03621U Infant Airway Management Trainer Head Only
LF00997U Adult Injectable Arm (Black)	LF03622U Intraosseous Infusion Right Leg
LF00999U Pediatric Injectable Head	LF03623U Infant Airway Management Trainer
LF01008U Intradermal Injection Arm	LF03626U Child Femoral Access Injection Pad Replacement
LF01012U Heart Catheterization (TPN)	LF03632U Child Intraosseous Infusion/ Femoral Access Leg on a Stand
LF01019U Ear Examination	LF03633U Child Airway Management Trainer with Torso
LF01020U Supplementary Ear Set	LF03693U Basic Buddy CPR Manikin
LF01025U Male Cath-Ed I	LF03699U "Airway Larry" Airway Management Trainer
LF01026U Female Cath-Ed II	LF03720U Baby Buddy Infant CPR Manikin
LF01027U Peritoneal Dialysis	LF03953U CRISIS ™ Manikin
LF01028U Suture Practice Arm	LF03955U Deluxe CRISIS ™ Manikin
LF01036U Spinal Injection	LF03965U Deluxe "Plus" CRISIS ™ Manikin
LF01037U Hemodialysis Arm	LF04001U GERI ™ Nursing Manikin
LF01062U Pelvic, Normal & Abnormal	LF04020U KERI ™ Nursing Manikin
LF01063U Stump Bandaging, Upper	LF04021U KERI ™ Basic Manikin
LF01064U Stump Bandaging, Lower	LF04022U KERI ™ Advanced Manikin
LF01069U Cervical Effacement	LF04030U GERI ™ Advanced Manikin
LF01070U Birthing Station	LF04040U GERI ™ Basic Manikin
LF01082U Cricothyrotomy	LF06001U CPR Prompt ™ Adult/Child Manikin
LF01083U Tracheostomy Care	LF06012U CPR Prompt ™ Infant Manikin
LF01084U Sigmoidoscopic Examination	LF06200U CPR Prompt ™ Keychain Rescue Aid
LF01087U Central Venous Cannulation	LF06204U CPR Prompt ™ Rescue and Practice Aid
LF01095U Blood Pressure Arm	
LF01108U Intraosseous Infusion Simulator	
LF01121U Advanced IV Arm	
LF01139U Advanced IV Hand	
LF01142U Auscultation Trainer	
LF01162U Venatech IV Trainer	
LF03000U CPARLENE ® Series	
LF03601U Adult Airway Management Trainer	
LF03602U Adult Airway Management on Manikin	
LF03603U Adult Airway Management Head Only	
LF03609U Child Airway Management Trainer	

Nasco Fort Atkinson

901 Janesville Avenue, P.O. Box 901
Fort Atkinson, Wisconsin 53538-0901
1-800-558-9595

www.eNasco.com • E-mail: lifeform@eNasco.com

COPYRIGHT © NASCO 1983 PRINTED IN U.S.A. NP 269-83/RV 8-05

Nasco
Life/form®

PELVIC EXAMINATION SIMULATOR LF01062U INSTRUCTION MANUAL



Life/form® Products by NASCO

About the Simulators...

The *Life/form*[®] Pelvic Exam Simulators are designed to facilitate the teaching, demonstrating, and practicing of pelvic examination for the detection of tumors. Every effort has been made to provide exceptional realism in the simulators so that the visual and tactile sensations experienced provide a positive transition to patient procedures.

Internal Structure

Each simulator was made from the same mold, and both are identical except for the palpable internal pathology. One of the simulators is considered “normal” with normal ovaries, cervix, and uterus. The other “abnormal” simulator has one ovary with a cyst and the other ovary has a solid tumor. The uterus has changes simulating a fibroid.

Procedures That Can Be Performed on These Simulators

Palpation and manipulation with hands and fingers in and on the internal and external genitalia, abdominal wall, ovaries, and uterus can be performed with very realistic tactile experience.

Bony landmarks and normal and atypical organs are anatomically correct in size, shape, texture, and location. The tumor and masses are texturally realistic and are accurately placed anatomically.

These simulators are intended for digital examination only. The use of a vaginal speculum is not intended or recommended.

General Instructions for Use

The use of gloves and liberal use of lubricant teach and reinforce good practices and are essential to protecting the internal surfaces of the simulators. The simulators must be treated like patients. Rough usage and lack of lubrication will generate tears and ruptures in the soft internal plastics.

CAUTION: Do not use any petroleum-based lubricants. Use of such lubricants will cause rapid deterioration and permanent damage. Use Lubricant Kit (LF00985U) or Ivory[®] liquid soap only.

Care of the Simulators

Store simulators in clean plastic bags (garbage bags). Soft plastic needed in manufacture does attract dust. Never place simulators on newsprint or any kind of printed paper or plastics. They will leave indelible stains. The simulators can be cleaned with mild soap (Ivory[®] liquid soap) and warm water. Use REN Cleaner (W09919U) to remove stubborn stains.

Supplies/Replacement Parts for the Pelvic Exam Simulators:

LF00985U	Lubricant Kit
W09919U	REN Cleaner

About the Simulators...

The *Life/form*[®] Pelvic Exam Simulators are designed to facilitate the teaching, demonstrating, and practicing of pelvic examination for the detection of tumors. Every effort has been made to provide exceptional realism in the simulators so that the visual and tactile sensations experienced provide a positive transition to patient procedures.

Internal Structure

Each simulator was made from the same mold, and both are identical except for the palpable internal pathology. One of the simulators is considered “normal” with normal ovaries, cervix, and uterus. The other “abnormal” simulator has one ovary with a cyst and the other ovary has a solid tumor. The uterus has changes simulating a fibroid.

Procedures That Can Be Performed on These Simulators

Palpation and manipulation with hands and fingers in and on the internal and external genitalia, abdominal wall, ovaries, and uterus can be performed with very realistic tactile experience.

Bony landmarks and normal and atypical organs are anatomically correct in size, shape, texture, and location. The tumor and masses are texturally realistic and are accurately placed anatomically.

These simulators are intended for digital examination only. The use of a vaginal speculum is not intended or recommended.

General Instructions for Use

The use of gloves and liberal use of lubricant teach and reinforce good practices and are essential to protecting the internal surfaces of the simulators. The simulators must be treated like patients. Rough usage and lack of lubrication will generate tears and ruptures in the soft internal plastics.

CAUTION: Do not use any petroleum-based lubricants. Use of such lubricants will cause rapid deterioration and permanent damage. Use Lubricant Kit (LF00985U) or Ivory[®] liquid soap only.

Care of the Simulators

Store simulators in clean plastic bags (garbage bags). Soft plastic needed in manufacture does attract dust. Never place simulators on newsprint or any kind of printed paper or plastics. They will leave indelible stains. The simulators can be cleaned with mild soap (Ivory[®] liquid soap) and warm water. Use REN Cleaner (W09919U) to remove stubborn stains.

Supplies/Replacement Parts for the Pelvic Exam Simulators:

LF00985U	Lubricant Kit
W09919U	REN Cleaner