



**DISPOSABLE IV
DEMONSTRATION ARM
LF01131U & LF01132U
INSTRUCTION MANUAL**



Life/form® Products by NASCO

About the Simulator...

The **Life/form**® Disposable IV Demonstration Arm may be used to practice venipuncture, IV placement, intramuscular injection, and suturing. Although this practice and demonstration arm is considered disposable, with proper care and treatment it will serve its purpose for a long period of time.

The life of the skin may be prolonged by utilizing smaller needle sizes (20- to 25-gauge recommended). Prepare the injection sites with distilled water only. You may inject ½ cc of distilled water, however, we recommend using air as the injectant since the water cannot be drained, but must evaporate from the arm. **NEVER** use synthetic blood for the injections.

The simulator can also be used for suturing demonstrations. “Wounds” may be cut at any desired depth or location with a sharp scalpel (Figure 1). Each “wound” may be sutured repeatedly, until the skin becomes so heavily perforated that it will no longer support more suture practice (Figure 2). The “wounds” can be “bandaged” with most standard adhesive bandages. To remove any residual adhesive, use only Glue-Off (LF01023U).



FIGURE 1

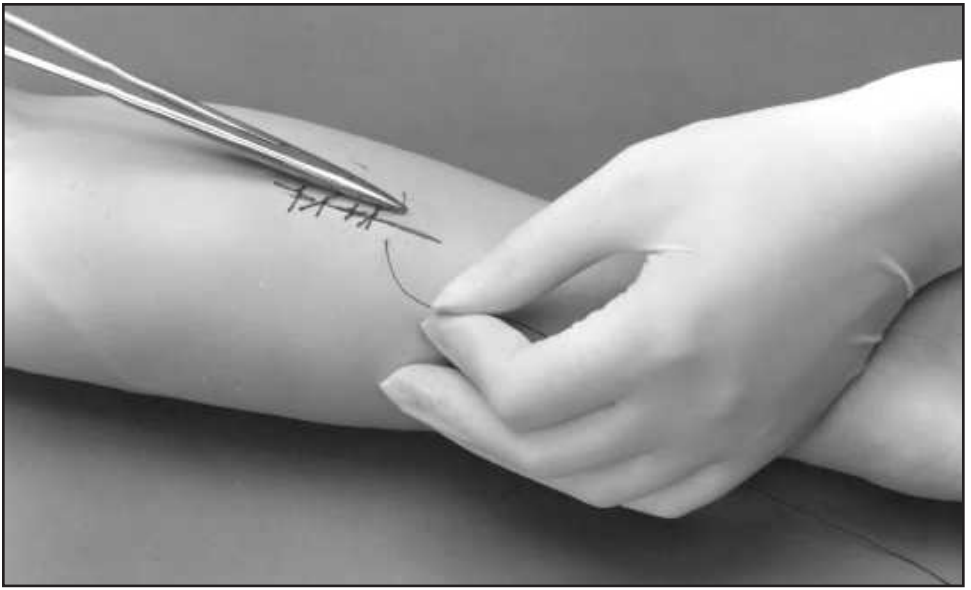


FIGURE 2

List of Components:

- Disposable IV Demonstration Arm
- 3 cc Syringe
- 22-Gauge Needle
- Suturing Set

Care of the Simulator:

1. Do not use povidone-iodine prep solutions, as they will indelibly stain the arm.
2. Clean normal soil with mild soap and water. Stubborn stains can be treated with REN Cleaner (W09919U).
3. Avoid any contact with ballpoint pen, newsprint, or other printed material, as these will permanently stain the simulator.

Other Available *Life/form*® Simulators

- LF00698U** Adult Injectable Arm (White)
LF00856U Female Catheterization
LF00901U Prostate Examination
LF00906U Ostomy Care
LF00929U Surgical Bandaging
LF00957U Enema Administration
LF00958U Pediatric Injectable Arm
LF00961U Intramuscular Injection
LF00984U Breast Examination
LF00995U Arterial Puncture Arm
LF00997U Adult Injectable Arm (Black)
LF00999U Pediatric Injectable Head
LF01008U Intradermal Injection Arm
LF01012U Heart Catheterization (TPN)
LF01019U Ear Examination
LF01020U Supplementary Ear Set
LF01025U Male Cath-Ed I
LF01026U Female Cath-Ed II
LF01027U Peritoneal Dialysis
LF01028U Suture Practice Arm
LF01036U Spinal Injection
LF01053U Cross-Sectional Anatomy, Torso, Head
LF01054U Cross-Sectional Anatomy, Head
LF01062U Pelvic, Normal & Abnormal
LF01063U Stump Bandaging, Upper
LF01064U Stump Bandaging, Lower
LF01069U Cervical Effacement
LF01070U Birthing Station
LF01082U Cricothyrotomy
LF01083U Tracheostomy Care
LF01084U Sigmoidoscopic Examination
LF01087U Central Venous Cannulation
LF01095U Blood Pressure Arm
LF01108U Intraosseous Infusion Simulator
LF01142U Auscultation Trainer
LF01162U Venatech IV Trainer
LF03000U **CPARLENE**® Series
LF03601U Adult Airway Management Trainer
LF03602U Adult Airway Management on Manikin
LF03603U Adult Airway Management Head Only
LF03609U Child Airway Management Trainer
LF03610U Child Airway Management Trainer Head Only
LF03611U Child Defibrillation Chest Skin
LF03612U Child IV Arm
LF03613U Child Blood Pressure Arm
LF03614U Child Intraosseous Infusion/ Femoral Access Leg Only
LF03615U Complete Child **CRiSis**™ Update Kit
LF03616U Child **CRiSis**™ Manikin
LF03617U Deluxe Child **CRiSis**™ Manikin with Arrhythmia Tutor
LF03620U PALS Update Kit
LF03621U Infant Airway Management Trainer Head Only
LF03622U Intraosseous Infusion Right Leg
LF03623U Infant Airway Management Trainer
LF03626U Child Femoral Access Injection Pad Replacement
LF03632U Child Intraosseous Infusion/ Femoral Access Leg on a Stand
LF03633U Child Airway Management Trainer with Torso
LF03693U **Basic Buddy** CPR Manikin
LF03699U "Airway Larry" Airway Management Trainer
LF03720U **Baby Buddy** Infant CPR Manikin
LF03953U **CRiSis**™ Manikin
LF03955U Deluxe **CRiSis**™ Manikin
LF04001U **GERi**™ Nursing Manikin
LF04020U **KERi**™ Nursing Manikin
LF04021U **KERi**™ Basic Manikin
LF04022U **KERi**™ Advanced Manikin
LF04030U **GERi**™ Advanced Manikin
LF04040U **GERi**™ Basic Manikin

Nasco Fort Atkinson

901 Janesville Avenue, P.O. Box 901
 Fort Atkinson, Wisconsin 53538-0901
 1-800-558-9595

www.eNASCO.com • E-mail: lifeform@eNASCO.com