

Nasco
Life/form®

**PATIENT EDUCATION
ADULT TRACHEOSTOMY
CARE SIMULATOR
LF01168U
INSTRUCTION MANUAL**



Life/form® Products by NASCO

About the Simulator...



The **Life/form**® Patient Education Tracheostomy Care Simulator is ideal for teaching patients and caregivers the skills they will need to perform at home. The adult manikin includes oral and nasal passages, the pharynx, epiglottis, trachea esophagus, tracheostomy site, and cricoid cartilage. The mouth and jaw are flexible to allow oral suctioning. The trachea branches into the left and right bronchial trees. Students can practice suctioning techniques, proper cuff inflation, dressing changes and other techniques. Cleansing and maintenance of the external tracheal site can be performed just as on an actual patient.

List of Components

1. Adult Tracheostomy Care Simulator
2. Adult Tracheostomy Tube
3. Carrying/Storage Case
4. NASCO **Life/form**® Lubricant

General Information about the Simulator

Procedures that may be performed on the Simulator:

- Oral-pharyngeal suctioning
- Nasal tracheal suctioning
- Tracheal suctioning via tracheostomy
- Dressing changes
- Stoma cleansing
- Changing of tracheostomy tube
- Cuff inflation
- Changing tracheostomy ties

Anatomy Present on the Simulator:

- Oral Passages
- Nasal Passages
- Pharynx
- Epiglottis
- Trachea
- Esophagus
- Stoma
- Right and Left Bronchi

The simulator may be used in all methods of tracheostomy care. Individual methods and procedures may differ; therefore this booklet does not propose to outline a particular procedure to be followed. Rather, it is recommended that a procedural manual and instructor's directions be followed. This will eliminate confusion and best serve the individual instructor and curriculum.

Care of the Simulator

Normal surface soil can be removed with a mild soap and luke-warm water. Use REN cleaner (W09919UA) to remove stubborn stains. Simply apply the REN cleaner to the soiled area and wipe with a clean, soft paper towel.

Cautions

1. Do not cleanse the stoma area with Betadine or any other similar, colored disinfectant. NEVER place the trainer on any kind of printed paper or plastic. These materials, as well as ballpoint pens, will transfer indelible stains. Do not use any cosmetics on the simulator.

2. With repeated use, the suctioning catheter will become soft and pliable, making it difficult to insert. You must either use a new catheter, or place the old one in a refrigerator, or on ice for a few minutes to stiffen it.

3. Heavy lubrication must be used while practicing procedures. The simulator will not function properly unless heavy lubrication is used. Use NASCO **Life/form**[®] Lubricant only. Do not use a water soluble lubricant.

4. Exercise the same care with this simulator as would be used with an actual patient. Excessive pressure and abuse will damage and shorten the units life.

**Supplies Replacement Parts
for the Patient Education Adult
Tracheostomy Care Simulator**

W09919UA REN Cleaner

LF00985UA Lubricant Kit

LF1170U Adult Trachea Tube

Other Available *Life/form*® Simulators

- LF00698U** Adult Injectable Arm (White)
LF00856U Female Catheterization
LF00901U Prostate Examination
LF00906U Ostomy Care
LF00929U Surgical Bandaging
LF00957U Enema Administration
LF00958U Pediatric Injectable Arm
LF00961U Intramuscular Injection
LF00984U Breast Examination
LF00995U Arterial Puncture Arm
LF00997U Adult Injectable Arm (Black)
LF00999U Pediatric Injectable Head
LF01008U Intradermal Injection Arm
LF01012U Heart Catheterization (TPN)
LF01019U Ear Examination
LF01020U Supplementary Ear Set
LF01025U Male Cath-Ed I
LF01026U Female Cath-Ed II
LF01027U Peritoneal Dialysis
LF01028U Suture Practice Arm
LF01036U Spinal Injection
LF01053U Cross-Sectional Anatomy, Torso, Head
LF01054U Cross-Sectional Anatomy, Head
LF01062U Pelvic, Normal & Abnormal
LF01063U Stump Bandaging, Upper
LF01064U Stump Bandaging, Lower
LF01069U Cervical Effacement
LF01070U Birthing Station
LF01082U Cricothyrotomy
LF01083U Tracheostomy Care
LF01084U Sigmoidoscopic Examination
LF01087U Central Venous Cannulation
LF01095U Blood Pressure Arm
LF01108U Intraosseous Infusion Simulator
LF01142U Auscultation Trainer
LF01162U Venatech IV Trainer
LF03000U **CPARLENE**® Series
LF03601U Adult Airway Management Trainer
LF03602U Adult Airway Management on Manikin
LF03603U Adult Airway Management Head Only
LF03609U Child Airway Management Trainer
LF03610U Child Airway Management Trainer Head Only
LF03611U Child Defibrillation Chest Skin
LF03612U Child IV Arm
LF03613U Child Blood Pressure Arm
LF03614U Child Intraosseous Infusion/Femoral Access Leg Only
LF03615U Complete Child **CRiSis**™ Update Kit
LF03616U Child **CRiSis**™ Manikin
LF03617U Deluxe Child **CRiSis**™ Manikin with Arrhythmia Tutor
LF03620U PALS Update Kit
LF03621U Infant Airway Management Trainer Head Only
LF03622U Intraosseous Infusion Right Leg
LF03623U Infant Airway Management Trainer
LF03626U Child Femoral Access Injection Pad Replacement
LF03632U Child Intraosseous Infusion/Femoral Access Leg on a Stand
LF03633U Child Airway Management Trainer with Torso
LF03693U **Basic Buddy** CPR Manikin
LF03699U "Airway Larry" Airway Management Trainer
LF03720U **Baby Buddy** Infant CPR Manikin
LF03953U **CRiSis**™ Manikin
LF03955U Deluxe **CRiSis**™ Manikin
LF04001U **GERi**™ Nursing Manikin
LF04020U **KERi**™ Nursing Manikin
LF04021U **GERi**™ Basic Manikin
LF04022U **KERi**™ Advanced Manikin
LF04030U **GERi**™ Advanced Manikin
LF04040U **GERi**™ Basic Manikin

Nasco Fort Atkinson

901 Janesville Avenue, P.O. Box 901
Fort Atkinson, Wisconsin 53538-0901
1-800-558-9595

www.eNASCO.com • E-mail: lifeform@eNASCO.com