



**INFANT AIRWAY
MANAGEMENT TRAINER
LF03621U
INSTRUCTION MANUAL**



Life/form® Products by NASCO

About the Simulator . . .

The **Life/form**® Infant Airway Management Trainer is the most realistic simulator available for the training of intubation skills. NASCO has taken great care to create an airway management trainer that is anatomically correct in respect to both size and detail.

Landmarks include: gum line, tongue, oral and nasal pharynx, larynx, epiglottis, arytenoids, false and true vocal cords, cricoid ring, tracheal rings, trachea, esophagus, lungs, and stomach. NASCO's Infant Airway Management Trainer allows you to practice oral intubation, and suction techniques can also be performed and evaluated. The simulator was designed to use an uncuffed endotracheal tube measuring up to 4.0 mm I.D.



LF03621U Infant Airway Management Trainer Head

List of Components:

1. Infant Airway Management Trainer Head
2. 8 oz. bottle of NASCO Lubricant

Set-Up:

Your Infant Airway Management Trainer is ready to use upon delivery. Simply remove it from the carton, and install it on the Resusci® Baby*.

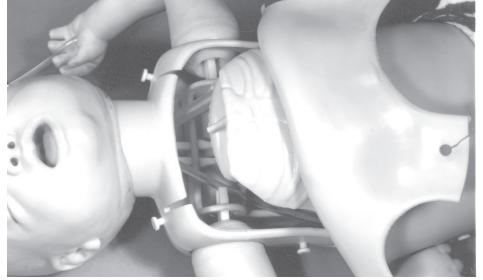


Figure 1

Removing the Resusci® Baby* Head:

Before it is possible to place the Infant Airway Management Trainer Head on the Resusci® Baby*, you will need to remove its present head. To do this, simply unhook the outer chest skin from the outer shell (Figure 1). Then disconnect the lung and stomach air lines where they connect into the head.

Once the lungs and stomach have been disconnected, remove the head from the body by squeezing together the "Z" tangs and sliding the head out from the body mounting bracket.

Now that the head has been removed, place the Infant Airway Management Trainer Head on the body. This can be done by lining up the "Z" tang on the neck of the head with the mounting opening in the body of the manikin. Once lined up, carefully push the "Z" tang into the opening until you hear it click into place (Figure 2).

***RESUSCI® BABY IS A TRADEMARK OF LAERDAL MEDICAL CORPORATION.**

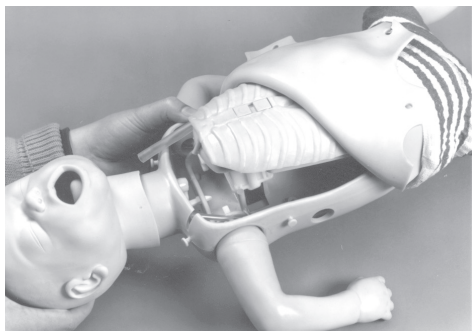


Figure 2

Connect the trachea tube up to the large diameter tubing and the esophagus tube to the small diameter tubing (**Note:** If no small diameter stomach tubing is present then you must plug off the esophagus. To do this simply loosen the clamp on the esophagus and remove the plastic fitting. Replace it with the small cap plug supplied with your kit and retighten the clamp). Place the outer chest skin back in place.



Figure 3

Lubrication:

Before using the Infant Airway Management Trainer, lubricate both the oral cavity and the endotracheal tube you plan to use with the NASCO lubricant provided (Figure 3).

***RESUSCI® BABY IS A TRADEMARK OF LAERDAL MEDICAL CORPORATION.**

PLEASE NOTE: NASCO recommends the use of the provided lubricant or a similar vegetable-based lubricant for the Infant Airway Management Trainer. The use of a silicone or similar lubricant may cause damage to the simulator, thus voiding NASCO's warranty on the trainer.

Cleaning the Face:

Normal soil can be removed from the face of the trainer with mild soapy water. REN Cleaner (W09919U) will remove stubborn stains. Simply apply REN to the soiled area and wipe clean with a soft cloth or paper towel.

IMPORTANT:

Never place the trainer on any kind of printed paper or plastic. These materials will transfer indelible stains. Ballpoint pens will also make indelible stains. Do not use any cosmetics.

Supplies/Replacement Parts for the Infant Airway Management Trainer

LF03644U NASCO Spray Lubricant

W09919U REN Cleaner

Other Available *Life/form* Simulators

- LF00698U** Adult Injectable Arm (White)
LF00856U Female Catheterization
LF00901U Prostate Examination
LF00906U Ostomy Care
LF00929U Surgical Bandaging
LF00957U Enema Administration
LF00958U Pediatric Injectable Arm
LF00961U Intramuscular Injection
LF00984U Breast Examination
LF00995U Arterial Puncture Arm
LF00997U Adult Injectable Arm (Black)
LF00999U Pediatric Injectable Head
LF01008U Intradermal Injection Arm
LF01012U Heart Catheterization (TPN)
LF01019U Ear Examination
LF01020U Supplementary Ear Set
LF01025U Male Cath-Ed I
LF01026U Female Cath-Ed II
LF01027U Peritoneal Dialysis
LF01028U Suture Practice Arm
LF01036U Spinal Injection
LF01037U Hemodialysis Arm
LF01062U Pelvic, Normal & Abnormal
LF01063U Stump Bandaging, Upper
LF01064U Stump Bandaging, Lower
LF01069U Cervical Effacement
LF01070U Birthing Station
LF01082U Cricothyrotomy
LF01083U Tracheostomy Care
LF01084U Sigmoidoscopic Examination
LF01087U Central Venous Cannulation
LF01095U Blood Pressure Arm
LF01108U Intraosseous Infusion Simulator
LF01121U Advanced IV Arm
LF01139U Advanced IV Hand
LF01142U Auscultation Trainer
LF01162U Venatech IV Trainer
LF03000U **CPARLENE**® Series
LF03601U Adult Airway Management Trainer
LF03602U Adult Airway Management on Manikin
LF03603U Adult Airway Management Head Only
LF03609U Child Airway Management Trainer
- LF03610U** Child Airway Management Trainer Head Only
LF03611U Child Defibrillation Chest Skin
LF03612U Child IV Arm
LF03613U Child Blood Pressure Arm
LF03614U Child Intraosseous Infusion/ Femoral Access Leg Only
LF03615U Complete Child **CRiSis**™ Update Kit
LF03616U Child **CRiSis**™ Manikin
LF03617U Deluxe Child **CRiSis**™ Manikin with Arrhythmia Tutor
LF03620U PALS Update Kit
LF03621U Infant Airway Management Trainer Head Only
LF03622U Intraosseous Infusion Right Leg
LF03623U Infant Airway Management Trainer
LF03626U Child Femoral Access Injection Pad Replacement
LF03632U Child Intraosseous Infusion/ Femoral Access Leg on a Stand
LF03633U Child Airway Management Trainer with Torso
LF03693U **Basic Buddy** CPR Manikin
LF03699U "Airway Larry" Airway Management Trainer
LF03720U **Baby Buddy** Infant CPR Manikin
LF03953U **CRiSis**™ Manikin
LF03955U Deluxe **CRiSis**™ Manikin
LF03965U Deluxe "Plus" **CRiSis**™ Manikin
LF04001U **GERi**™ Nursing Manikin
LF04020U **KERi**™ Nursing Manikin
LF04021U **KERi**™ Basic Manikin
LF04022U **KERi**™ Advanced Manikin
LF04030U **GERi**™ Advanced Manikin
LF04040U **GERi**™ Basic Manikin
LF06001U **CPR Prompt**™ Adult/Child Manikin
LF06012U **CPR Prompt**™ Infant Manikin
LF06200U **CPR Prompt**™ Keychain Rescue Aid
LF06200U **CPR Prompt**™ Rescue and Practice Aid

Nasco Fort Atkinson

901 Janesville Avenue, P.O. Box 901
Fort Atkinson, Wisconsin 53538-0901
1-800-558-9595

www.eNASCO.com • E-mail: lifeform@eNASCO.com