

CHILD CRISIS™ AIRWAY MANAGEMENT TRAINER LF03633U INSTRUCTION MANUAL



About the Simulator...

The *Life/form*_® Child *CRiSis*™ Airway Management Trainer Torso is the most realistic simulator available for the training of intubation and other airway management skills.

NASCO has taken great care to create an airway management trainer that is anatomically correct in respect to both size and detail. Landmarks include: nostrils, teeth, tongue, oral and nasal pharynx, larynx, epiglottis, arytenoids, false and true cords, trachea, esophagus, lungs, and stomach.

The NASCO Child Airway Management Trainer allows you to practice oral, digital, and nasal intubation, as well as suction techniques. The simulator was designed to use an uncuffed endotracheal tube measuring up to 5.0 mm I.D. With proper care, your *Life/form*® Child Airway Management Trainer will provide years of valuable service.

List of Components:

- 1. Child Airway Management Trainer with Torso
- 2. 4 oz. Can of Aerosol Lubricant

Set-Up:

Your Child Airway Management Trainer is ready to use upon delivery. Simply remove from carton and unwrap packaging material.



Figure 1



Figure 2 Lubrication:

Before intubating the Child Airway Management Trainer, lubricate both the simulator and tube with the NASCO lubricant provided (Figures 1 & 2).

Cleaning the Face:

Normal soil can be removed from the face of the trainer with mild soapy water. REN Cleaner (W09919U) will remove stubborn stains. Simply apply REN to soiled area and wipe clean with a soft cloth or paper towel. **NOTE:** Do not use REN Cleaner around the mouth and nostrils of head, as the residue of cleaner could be toxic.

Cautions:

NASCO recommends the use of the enclosed aerosol lubricant or a similar vegetable-based lubricant for use with our Child Airway Management Trainer.

DO NOT use a silicone or similar lubricant. This will cause your simulator to dry and crack, and will automatically void NASCO's warranty on the trainer.

Never place the trainer on any kind of printed paper or plastic. These materials will transfer indelible stains. Ballpoint pens will also make indelible stains.

Available Supplies:

LF03608U NASCO Lubricant

LF03627U Replacement Lung Set

LF03628U Replacement Stomach

LF03631U Replacement Headskin with airway

W09919U REN Cleaner

Add-On Components:

LF03611U Defibrillation Chest

 $\textbf{LF03612U} \hspace{0.2cm} \textbf{Left Arm} - \textbf{IV Training}$

LF03613U Right Arm — Blood

Pressure Monitoring

LF03614U Right Leg —

Intraosseous Infusion/Femoral

Access

LF03634U Left Leg

(Nonfunctioning)

Other Available *Life/form*_® Simulators

Other A	Available <i>Life/form</i> 。	Simulators	
LF00698U	Adult Injectable Arm (White)	LF03610U	Child Airway Management
	Female Catheterization		Trainer Head Only
LF00901U	Prostate Examination	LF03611U	Child Defibrillation Chest
LF00906U	Ostomy Care		Skin
LF00929U	Surgical Bandaging		Child IV Arm
	Enema Administration		Child Blood Pressure Arm
	Pediatric Injectable Arm	LF03614U	Child Intraosseous Infusion/
	Intramuscular Injection		Femoral Access Leg Only
	Breast Examination	LF03615U	Complete Child CRISIS™
	Arterial Puncture Arm		Update Kit
LF00997U	Adult Injectable Arm (Black)		Child CRISis™ Manikin
LF00999U	Pediatric Injectable Head	LF03617U	Deluxe Child CRISIS™
	Intradermal Injection Arm	1502/0011	Manikin with Arrhythmia Tutor
	Heart Catheterization (TPN) Ear Examination	LF03620U	PALS Update Kit
	Supplementary Ear Set	LFU3021U	Infant Airway Management
LF010200	Male Cath-Ed I	1 E0242211	Trainer Head Only Intraosseous Infusion Right
	Female Cath-Ed II	LF03022 0	Leg
	Peritoneal Dialysis	LE0362311	Infant Airway Management
	Suture Practice Arm	LI 000200	Trainer
LF01026U	Spinal Injection	LF03626U	Child Femoral Access
LF01037U	Hemodialysis Arm	E1 000200	Injection Pad Replacement
LF01062U	Pelvic, Normal & Abnormal	LF03632U	Child Intraosseous Infusion/
LF01063U	Stump Bandaging, Upper	1.000020	Femoral Access Leg on a
LF01064U	Stump Bandaging, Lower		Stand
LF01069U	Cervical Effacement	LF03633U	Child Airway Management
LF01070U	Birthing Station		Trainer with Torso
LF01082U	Cricothyrotomy	LF03693U	Basic Buddy CPR Manikin
LF01083U	Tracheostomy Care	LF03699U	"Airway Larry" Airway
LF01084U	Sigmoidoscopic		Management Trainer
	Examination		Baby Buddy Infant CPR Manikin
	Central Venous Cannulation	LF03953U	CRISis™ Manikin
	Blood Pressure Arm	LF03955U	Deluxe CRISIS™ Manikin
LF01108U	Intraosseous Infusion		Deluxe "Plus" CRISIS™ Manikin
	Simulator	LF04001U	GERi™ Nursing Manikin
	Advanced IV Arm	LF04020U	KERI [™] Nursing Manikin
	Advanced IV Hand		KERI [™] Basic Manikin
	Auscultation Trainer	LF04022U	KERi™ Advanced Manikin GERi™ Advanced Manikin
	Venatech IV Trainer CPARLENE® Series	LF04030U	GERi™ Advancea Manikin GERi™ Basic Manikin
LF030000	Adult Airway Management		CPR Prompt TM Adult/Child
LF030010	Trainer	LFUOUUIU	Manikin
1 E0360211	Adult Airway Management	LE0601211	CPR Prompt™ Infant Manikin
L1 00002 0	on Manikin	[F06200120	CPR Prompt™ Keychain
LF03603U	Adult Airway Management	E1 002000	Rescue Aid
<u></u>	Head Only	LF06200U	CPR Prompt™ Rescue and
LF03609U	Child Airway Management		Practice Aid
	Trainer		
	=		



901 Janesville Avenue, P.O. Box 901 Fort Atkinson, Wisconsin 53538-0901 1-800-558-9595