



---

**INFANT INJECTABLE  
TRAINING LEG REPLACEMENT  
SKIN AND VEINS  
LF03639U & LF03640U  
INSTRUCTION MANUAL**

---



**Life/form**® Products by NASCO

## List of Components:

1. Replacement Vein Sections (10)
2. Replacement Skin (Not Included with LF03640U Vein Replacement Kit)
3. Lubricating Powder
4. Liquid Lubricant

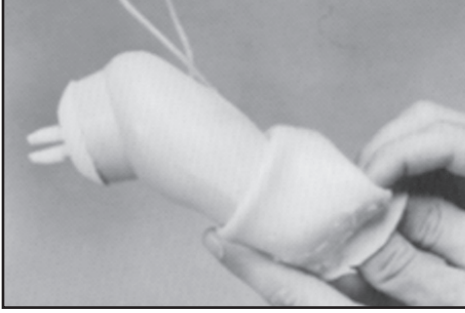


Figure 1

## Removing the Used Skin From the Leg:

1. Lubricate the exterior of the skin with the dry lubricating powder.
2. Peel the skin off by turning it inside out (Figure 1).
3. Remove the old tubing.

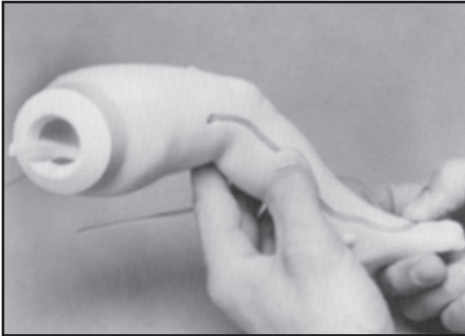


Figure 2

## Installing the New Tubing:

Place the new section of tubing in the channel of the leg core (Figure 2).

Be certain that equal portions of the tubing are protruding from the leg.

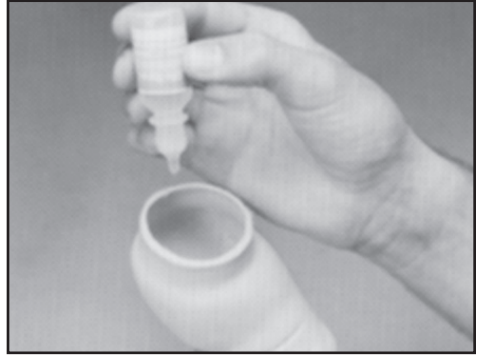


Figure 3

## Lubricating the Skin:

Pour only enough of the *Life/form*<sup>®</sup> Lubricant (provided) to coat all inside surfaces of the skin (Figure 3).

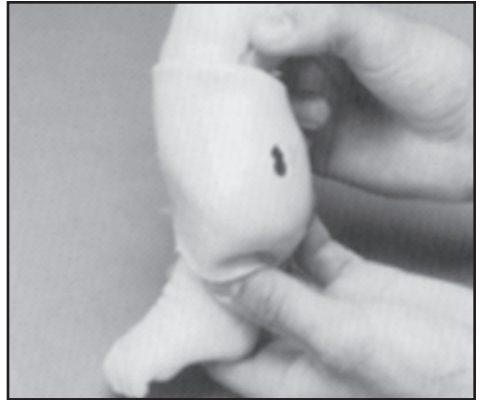
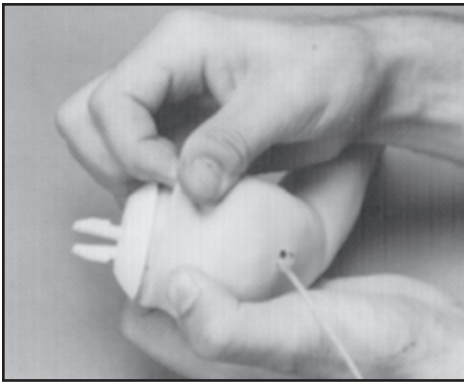


Figure 4

## Installing the New Skin:

1. Slide the hip of the skin over the foot of the core (Figure 4). Be certain to align the ankles of the core and skin.
2. Grasp the skin with both hands and slide it over the core until the toes of the core approach



**Figure 5**

3. Check the tubing location. If the tubing has moved from the channel, work it back into place by palpating it through the skin. If unsuccessful, peel the skin back (invert) until the tubing can be placed properly in the channel and then roll the skin back over the core.
4. Work the toes into place.
5. Draw the skin snugly into place (Figure 5).

### **Rinse:**

Wash the excess lubricant from the exterior of the leg skin. The Infant Injectable Training Leg Simulator is now fully reconditioned and ready for continued training.

## **Supplies/Replacement Parts for the Infant Injectable Training Leg Simulator:**

- LF03639U** Infant Leg Skin (1) and Vein (10) Replacement Kit
- LF03640U** Infant Leg Vein Replacement Kit (10)
- LF01099U** Vein Tubing Sealant Kit
- LF00845U** *Life/form*® Venous Blood, 1 Quart
- LF00846U** *Life/form*® Venous Blood, 1 Gallon
- LF01022U** Fluid Supply Stand
- LF01130U** Intravenous Fluid Bag, 500 ml
- W09919U** REN Cleaner

## Other Available *Life/form* Simulators

- LF00698U** Adult Injectable Arm (White)  
**LF00856U** Female Catheterization  
**LF00901U** Prostate Examination  
**LF00906U** Ostomy Care  
**LF00929U** Surgical Bandaging  
**LF00957U** Enema Administration  
**LF00958U** Pediatric Injectable Arm  
**LF00961U** Intramuscular Injection  
**LF00984U** Breast Examination  
**LF00995U** Arterial Puncture Arm  
**LF00997U** Adult Injectable Arm (Black)  
**LF00999U** Pediatric Injectable Head  
**LF01008U** Intradermal Injection Arm  
**LF01012U** Heart Catheterization (TPN)  
**LF01019U** Ear Examination  
**LF01020U** Supplementary Ear Set  
**LF01025U** Male Cath-Ed I  
**LF01026U** Female Cath-Ed II  
**LF01027U** Peritoneal Dialysis  
**LF01028U** Suture Practice Arm  
**LF01036U** Spinal Injection  
**LF01037U** Hemodialysis Arm  
**LF01062U** Pelvic, Normal & Abnormal  
**LF01063U** Stump Bandaging, Upper  
**LF01064U** Stump Bandaging, Lower  
**LF01069U** Cervical Effacement  
**LF01070U** Birthing Station  
**LF01082U** Cricothyrotomy  
**LF01083U** Tracheostomy Care  
**LF01084U** Sigmoidoscopic Examination  
**LF01087U** Central Venous Cannulation  
**LF01095U** Blood Pressure Arm  
**LF01108U** Intraosseous Infusion Simulator  
**LF01121U** Advanced IV Arm  
**LF01139U** Advanced IV Hand  
**LF01142U** Auscultation Trainer  
**LF01162U** Venatech IV Trainer  
**LF03000U** **CPARLENE**® Series  
**LF03601U** Adult Airway Management Trainer  
**LF03602U** Adult Airway Management on Manikin  
**LF03603U** Adult Airway Management Head Only  
**LF03609U** Child Airway Management Trainer  
**LF03610U** Child Airway Management Trainer Head Only  
**LF03611U** Child Defibrillation Chest Skin  
**LF03612U** Child IV Arm  
**LF03613U** Child Blood Pressure Arm  
**LF03614U** Child Intraosseous Infusion/ Femoral Access Leg Only  
**LF03615U** Complete Child **CRiSis**™ Update Kit  
**LF03616U** Child **CRiSis**™ Manikin  
**LF03617U** Deluxe Child **CRiSis**™ Manikin with Arrhythmia Tutor  
**LF03620U** PALS Update Kit  
**LF03621U** Infant Airway Management Trainer Head Only  
**LF03622U** Intraosseous Infusion Right Leg  
**LF03623U** Infant Airway Management Trainer  
**LF03626U** Child Femoral Access Injection Pad Replacement  
**LF03632U** Child Intraosseous Infusion/ Femoral Access Leg on a Stand  
**LF03633U** Child Airway Management Trainer with Torso  
**LF03693U** **Basic Buddy** CPR Manikin  
**LF03699U** "Airway Larry" Airway Management Trainer  
**LF03720U** **Baby Buddy** Infant CPR Manikin  
**LF03953U** **CRiSis**™ Manikin  
**LF03955U** Deluxe **CRiSis**™ Manikin  
**LF03965U** Deluxe "Plus" **CRiSis**™ Manikin  
**LF04001U** **GERi**™ Nursing Manikin  
**LF04020U** **KERi**™ Nursing Manikin  
**LF04021U** **KERi**™ Basic Manikin  
**LF04022U** **KERi**™ Advanced Manikin  
**LF04030U** **GERi**™ Advanced Manikin  
**LF04040U** **GERi**™ Basic Manikin  
**LF06001U** **CPR Prompt**™ Adult/Child Manikin  
**LF06012U** **CPR Prompt**™ Infant Manikin  
**LF06200U** **CPR Prompt**™ Keychain Rescue Aid  
**LF06200U** **CPR Prompt**™ Rescue and Practice Aid

*Nasco* Fort Atkinson

901 Janesville Avenue, P.O. Box 901  
Fort Atkinson, Wisconsin 53538-0901  
1-800-558-9595

www.eNASCO.com • E-mail: lifeform@eNASCO.com