




**INFANT INJECTABLE
TRAINING ARM REPLACEMENT
SKIN AND VEINS
LF03641U
INSTRUCTION MANUAL**

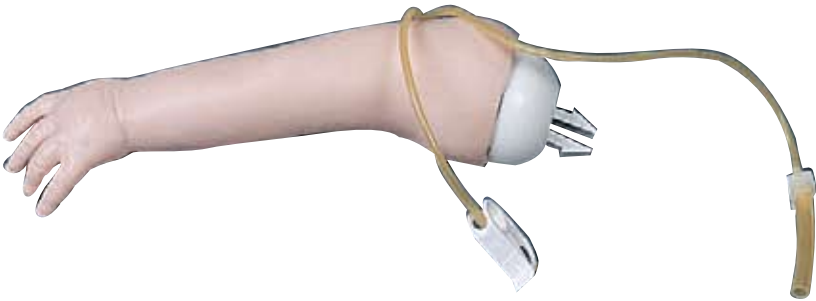

WARNING: Product contains dry natural rubber.

Life/form® Products by NASCO

List of Components:

1. Replacement Skin
2. Replacement Vein Section
3. Lubricating Powder
4. Liquid Lubricant

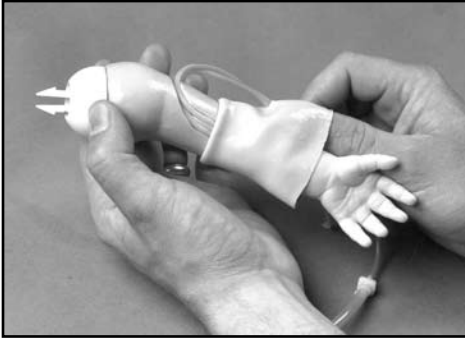


Figure 1

Removing the Used Skin from the Arm:

1. Lubricate the exterior of the skin with baby powder.
2. Peel the skin off by turning it inside out (Figure 1).
3. Remove the old tubing.

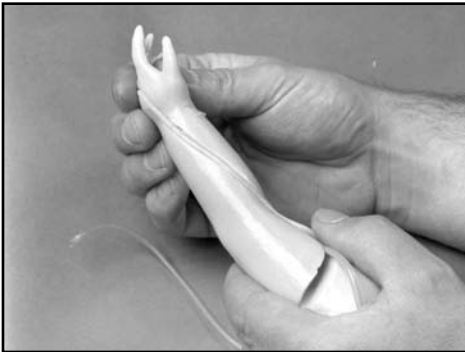


Figure 2

Installing the New Tubing:

Place the new section of tubing into the channel of the arm core (Figure 2). Be certain equal portions of tubing protrude at the shoulder.

1



Figure 3

Lubricating the Skin:

Pour only enough of the liquid *Life/form*[®] Lubricant provided into the arm skin to coat all the inside surfaces (Figure 3).



Figure 4

Installing the New Skin:

1. Slide the shoulder of the skin over the hand of the core (Figure 4). Be certain to align the palm of the core and skin.
2. Grasp the skin and slide it over the core until the fingers of the core approach the finger holes of the skin.

CAUTION: The thin *Life/form*[®] skin used to provide realism is fragile and care must be taken not to stretch or tear the material when installing.



Figure 5

3. Check the tubing location. If the tubing has moved from the channel, work it back into place by palpating it through the skin (Figure 5). If unsuccessful, peel the skin back (invert) until the tubing can be replaced properly and then roll the skin back over the core.
4. Pull the tubing ends through the hole in the side of the arm skin.
5. Work the fingers into place.
6. Draw the skin snugly into place.

Supplies/Replacement Parts for the Infant Injectable Training Arm Skin and Vein:

- LF00845U** *Life/form*® Venous Blood, 1 quart
- LF00846U** *Life/form*® Venous Blood, 1 gallon
- LF03641U** Skin and Vein Replacement Kit
- LF01099U** Vein Tubing Sealant Kit
- LF01022U** Fluid Supply Stand
- LF01130U** Intravenous Fluid Bag, 500 ml
- W09919U** REN Cleaner

Other Available *Life/form* Simulators

- LF00698U** Adult Injectable Arm (White)
LF00856U Female Catheterization
LF00901U Prostate Examination
LF00906U Ostomy Care
LF00929U Surgical Bandaging
LF00957U Enema Administration
LF00958U Pediatric Injectable Arm
LF00961U Intramuscular Injection
LF00984U Breast Examination
LF00995U Arterial Puncture Arm
LF00997U Adult Injectable Arm (Black)
LF00999U Pediatric Injectable Head
LF01008U Intradermal Injection Arm
LF01012U Heart Catheterization (TPN)
LF01019U Ear Examination
LF01020U Supplementary Ear Set
LF01025U Male Cath-Ed I
LF01026U Female Cath-Ed II
LF01027U Peritoneal Dialysis
LF01028U Suture Practice Arm
LF01036U Spinal Injection
LF01037U Hemodialysis Arm
LF01062U Pelvic, Normal & Abnormal
LF01063U Stump Bandaging, Upper
LF01064U Stump Bandaging, Lower
LF01069U Cervical Effacement
LF01070U Birthing Station
LF01082U Cricothyrotomy
LF01083U Tracheostomy Care
LF01084U Sigmoidoscopic Examination
LF01087U Central Venous Cannulation
LF01095U Blood Pressure Arm
LF01108U Intraosseous Infusion Simulator
LF01121U Advanced IV Arm
LF01139U Advanced IV Hand
LF01142U Auscultation Trainer
LF01162U Venatech IV Trainer
LF03000U **CPARLENE**® Series
LF03601U Adult Airway Management Trainer
LF03602U Adult Airway Management on Manikin
LF03603U Adult Airway Management Head Only
LF03609U Child Airway Management Trainer
LF03610U Child Airway Management Trainer Head Only
LF03611U Child Defibrillation Chest Skin
LF03612U Child IV Arm
LF03613U Child Blood Pressure Arm
LF03614U Child Intraosseous Infusion/Femoral Access Leg Only
LF03615U Complete Child **CRiSis**™ Update Kit
LF03616U Child **CRiSis**™ Manikin
LF03617U Deluxe Child **CRiSis**™ Manikin with Arrhythmia Tutor
LF03620U PALS Update Kit
LF03621U Infant Airway Management Trainer Head Only
LF03622U Intraosseous Infusion Right Leg
LF03623U Infant Airway Management Trainer
LF03626U Child Femoral Access Injection Pad Replacement
LF03632U Child Intraosseous Infusion/Femoral Access Leg on a Stand
LF03633U Child Airway Management Trainer with Torso
LF03693U **Basic Buddy** CPR Manikin
LF03699U "Airway Larry" Airway Management Trainer
LF03720U **Baby Buddy** Infant CPR Manikin
LF03953U **CRiSis**™ Manikin
LF03955U Deluxe **CRiSis**™ Manikin
LF03965U Deluxe "Plus" **CRiSis**™ Manikin
LF04001U **GERi**™ Nursing Manikin
LF04020U **KERi**™ Nursing Manikin
LF04021U **KERi**™ Basic Manikin
LF04022U **KERi**™ Advanced Manikin
LF04030U **GERi**™ Advanced Manikin
LF04040U **GERi**™ Basic Manikin
LF06001U **CPR Prompi**™ Adult/Child Manikin
LF06012U **CPR Prompi**™ Infant Manikin
LF06200U **CPR Prompi**™ Keychain Rescue Aid
LF06200U **CPR Prompi**™ Rescue and Practice Aid

Nasco Fort Atkinson

901 Janesville Avenue, P.O. Box 901
Fort Atkinson, Wisconsin 53538-0901
1-800-558-9595

www.eNASCO.com • E-mail: lifeform@eNASCO.com