

Nasco
Life/form®

**"Airway Larry" TORSO
MANIKIN
LF03669U
INSTRUCTION MANUAL**




Life/form® Products by NASCO

Limited Warranty

NASCO warrants to the purchasers of **CRiSis™** products that they will be free from defects in material and workmanship for a period of three years from the date of purchase. NASCO will repair any defect reported within three years of the date of purchase at no charge. Products found to be defective may be returned to the authorized NASCO dealer from whom the item was purchased, or returned directly to NASCO. NASCO will be liable under this limited warranty only if **CRiSis™** products have been serviced properly as directed in the operating manual.

NASCO will not be responsible for damage caused by unauthorized repairs or modifications that have been made, or if the product has been damaged through misuse, accident, or abuse. This warranty does not cover wear and tear or expendables such as batteries, lubricant, and replacement lungs. There are no other expressed or implied warranties of merchantability, fitness of purpose, or otherwise on "Airway Larry" products, parts, and accessories.

Table of Contents	page
Introduction	2
Airway Management Trainer	3

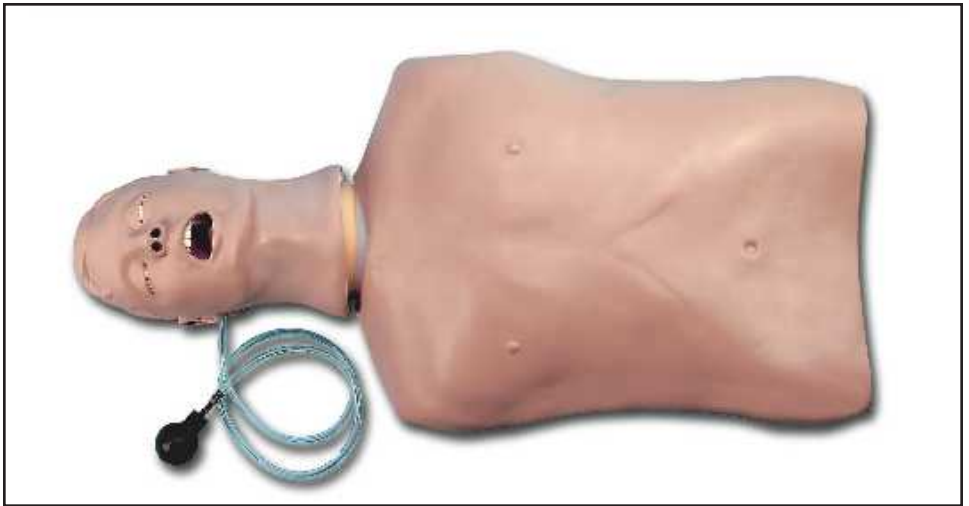

The "Airway Larry" Manikin is a Complete Resuscitation System consisting of modular components which allow you to create a manikin to suit your changing needs. The components may be purchased as a complete package or separately to update your existing manikin. Update packages are compatible with all versions of both **CPARLENE**[®] and Resusci[®] Anne[™] *.

This manual will guide you in setting up, using, and maintaining each of the available components. Each section also includes a list of replacement parts, supplies, and auxiliary equipment.

By reading and following all instructions carefully and completely, you can be sure your **Life/form**[®] "Airway Larry" Manikin will provide years of valuable service.

Cleaning:


Normal surface soil can be removed from the trainer with mild soapy water. Stubborn stains may be removed with REN Cleaner (W09919U). Simply apply the REN Cleaner to the soiled area and wipe clean with a soft cloth.

NOTE: Avoid using cleaner around the mouth area if students will be applying direct mouth-to-mouth resuscitation techniques, as the cleaner may be toxic if ingested. NEVER place the trainer on any kind of printed paper or plastic. These materials, as well as ballpoint pens, will transfer indelible stains. Do not use any cosmetics.

List of Components:

- "Airway Larry" Torso Manikin
- 8-oz. Pump Spray Lubricant
- Small Towel

*Resusci[®] Anne[™] is a trademark of Laerdal Medical Corporation.

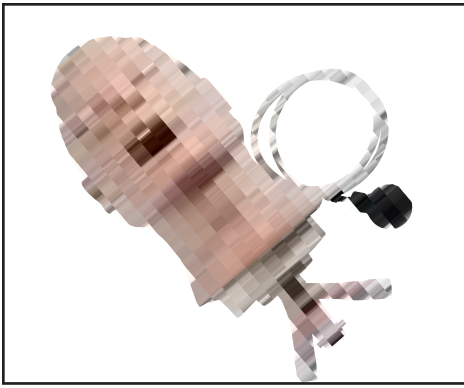

NASCO Life/form® Airway Management Head

About the Simulator...

The **Life/form®** Airway Management Trainer Head is the most realistic simulator available for the training of intubation and other airway management skills.

NASCO has taken great care to create an airway management trainer that is anatomically correct in respect to both size and detail. Landmarks include: nostrils, teeth, tongue, oral and nasal pharynx, larynx, cricoid ring, epiglottis, arytenoid, false and true cords, trachea, esophagus, "Airway Larry" lung set, and stomach.

NASCO's Airway Management Trainer Head allows you to practice oral, digital, and nasal intubation. E.T., E.O.A., PTL®, and Combitube® insertion can all be practiced as well (please see "Using the Combitube®"). Suction techniques and proper cuff inflation can also be performed and evaluated.

Lubrication:

Before using the Airway Management Trainer Torso, lubricate both the simulator and supplies being used with the NASCO lubricant provided (Figures 1 & 2).

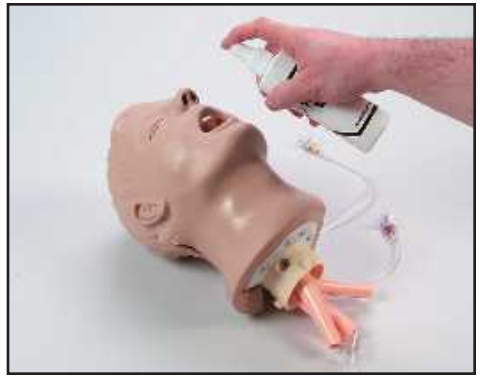


Figure 1



Figure 2

NOTE: NASCO recommends the use of the provided lubricant or a similar vegetable-based lubricant for the Airway Larry Management Trainer Head. The use of a silicone-based lubricant may cause damage to the simulator, thus voiding NASCO'S warranty on the trainer.

Setup:

The Airway Management Trainer Torso is ready to use upon delivery. Simply remove it from the carton, and unwrap the packaging material.

Cleaning & Maintenance

To clean the Airway Management Trainer head you will need to remove the head from the manikin. To do this first remove the chest skin and then the chest plate exposing the lungs (Figure 3).



Figure 3

Feed the lungs through the hole in the upper compression plate and then lift off the plate. Disconnect the lungs from the bronchi by pulling the red caps from the white connectors (Figure 4).



Figure 4

Now that the head has been disconnected from the airway rotate it 180° so that it is facing backwards. The large tab on the front of the neck should be aligned with the kiwi in the torso. Tilt the head upward until it snaps free (Figure 5). Disengage the smaller rear tabs from the neck opening and pull the head from the body. Reverse the above steps to reattach the head.



Figure 5

Once the head is off the body, place a cap plug (supplied) on each end branch of the bronchi (Figure 6) (the esophagus already has one in place). You are now ready to begin the cleaning process. Take the trainer to an area with a sink and open counter space.



Figure 6

Stabilize the head on the counter face-up (towels work well for this) with the plugged tubes hanging over the sink. Carefully pour warm soapy water (a mild dish soap works best) into the mouth until the water level reaches halfway up the tongue. Now tilt the head back and bring the neck up 3" off the countertop.

Continue filling until the water level covers the tongue. At this point, take a small soft brush and gently scrub the inside of the mouth. (A small toothbrush works well for this.) Cotton swabs can be used to scrub inside the nostrils. When done, pull the plug from the esophagus and drain into the sink. Now pick the head up to a vertical position and pull the plug from the trachea to completely empty the system (Figure 7).



Figure 7

Rinsing the Airway:

To rinse the airway, follow the same procedure using clean, warm tap water. Repeat this process until all the soap has been flushed from the system.

Disinfecting the Airway:

To disinfect, repeat the standard cleaning procedure, but this time use a bleach solution, as specified by the Centers for Disease Control, instead of soapy water. Fill the system with the solution until it reaches the corners of the mouth. Remember to start filling with the head flat and finish with the neck slightly elevated to ensure that the solution completely fills all airway passages.

Once completely filled with the bleach solution, allow the head to sit for at least 10 minutes. Drain as described earlier and repeat the rinsing process to flush out all of the bleach solution. Set the head aside and allow it to dry completely.

Fill the system with the solution until it reaches the corners of the mouth. Remember to start filling with the head flat and finish with the neck slightly elevated to ensure that the solution completely fills the airway passages. Once completely filled with the bleach solution, allow the head to sit for at least 10 minutes. Drain as described earlier and repeat the rinsing process to flush out all of the bleach solution. Set the head aside and allow it to dry completely.

Using the Combitube:

Thoroughly read and follow the instructions that come with the Combitube. The trainer will accept either a full size or a small adult tube. As with a live patient, it may be necessary to back the tube out slightly if ventilation cannot be established.

Note:

Depending on tube placement, the large cuff may not accept the recommended amount of air. In this case, simply inflate the cuff to its maximum volume (when the plunger stops) and, detach the syringe from the blue pilot balloon, and proceed.

Supplies/Replacement Parts for Airway Management Trainer Head:

- LF03285U Replacement Lungs
- LF03644U NASCO Pump Spray Lubricant
- W09919U REN Cleaner

Other Available *Life/form* Simulators

- LF00698U** Adult Injectable Arm (White)
LF00856U Female Catheterization
LF00901U Prostate Examination
LF00906U Ostomy Care
LF00929U Surgical Bandaging
LF00957U Enema Administration
LF00958U Pediatric Injectable Arm
LF00961U Intramuscular Injection
LF00984U Breast Examination
LF00995U Arterial Puncture Arm
LF00997U Adult Injectable Arm (Black)
LF00999U Pediatric Injectable Head
LF01008U Intradermal Injection Arm
LF01012U Heart Catheterization (TPN)
LF01019U Ear Examination
LF01020U Supplementary Ear Set
LF01025U Male Cath-Ed I
LF01026U Female Cath-Ed II
LF01027U Peritoneal Dialysis
LF01028U Suture Practice Arm
LF01036U Spinal Injection
LF01037U Hemodialysis Arm
LF01062U Pelvic, Normal & Abnormal
LF01063U Stump Bandaging, Upper
LF01064U Stump Bandaging, Lower
LF01069U Cervical Effacement
LF01070U Birthing Station
LF01082U Cricothyrotomy
LF01083U Tracheostomy Care
LF01084U Sigmoidoscopic Examination
LF01087U Central Venous Cannulation
LF01095U Blood Pressure Arm
LF01108U Intraosseous Infusion Simulator
LF01121U Advanced IV Arm
LF01139U Advanced IV Hand
LF01142U Auscultation Trainer
LF01162U Venatech IV Trainer
LF03000U **CPARLENE**® Series
LF03601U Adult Airway Management Trainer
LF03602U Adult Airway Management on Manikin
LF03603U Adult Airway Management Head Only
LF03609U Child Airway Management Trainer
- LF03610U** Child Airway Management Trainer Head Only
LF03611U Child Defibrillation Chest Skin
LF03612U Child IV Arm
LF03613U Child Blood Pressure Arm
LF03614U Child Intraosseous Infusion/ Femoral Access Leg Only
LF03615U Complete Child **CRiSis**™ Update Kit
LF03616U Child **CRiSis**™ Manikin
LF03617U Deluxe Child **CRiSis**™ Manikin with Arrhythmia Tutor
LF03620U PALS Update Kit
LF03621U Infant Airway Management Trainer Head Only
LF03622U Intraosseous Infusion Right Leg
LF03623U Infant Airway Management Trainer
LF03626U Child Femoral Access Injection Pad Replacement
LF03632U Child Intraosseous Infusion/ Femoral Access Leg on a Stand
LF03633U Child Airway Management Trainer with Torso
LF03693U **Basic Buddy** CPR Manikin
LF03699U "Airway Larry" Airway Management Trainer
LF03720U **Baby Buddy** Infant CPR Manikin
LF03953U **CRiSis**™ Manikin
LF03955U Deluxe **CRiSis**™ Manikin
LF03965U Deluxe "Plus" **CRiSis**™ Manikin
LF04001U **GERi**™ Nursing Manikin
LF04020U **KERi**™ Nursing Manikin
LF04021U **KERi**™ Basic Manikin
LF04022U **KERi**™ Advanced Manikin
LF04030U **GERi**™ Advanced Manikin
LF04040U **GERi**™ Basic Manikin
LF06001U **CPR Prompt**™ Adult/Child Manikin
LF06012U **CPR Prompt**™ Infant Manikin
LF06200U **CPR Prompt**™ Keychain Rescue Aid
LF06200U **CPR Prompt**™ Rescue and Practice Aid

Nasco Fort Atkinson

901 Janesville Avenue, P.O. Box 901
Fort Atkinson, Wisconsin 53538-0901
1-800-558-9595

www.eNASCO.com • E-mail: lifeform@eNASCO.com