



CPR Savers
& FIRST AID SUPPLY®

1-800-480-1277

How to Replace the 'Clicker' of the Prestan Professional Manikin

*** Tool required: a small slotted style screwdriver**

- Turn the manikin over so that the chest of the manikin is down and the back of the manikin is facing you.
Note the 3 parts shown in Figure #1.

Rate Monitor

Wire Cover

Clicker

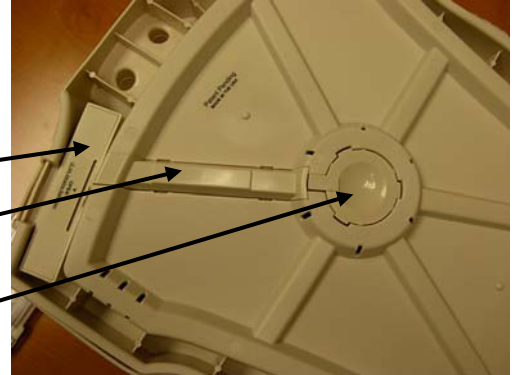


Figure #1

- **1) Remove the Wire Cover** – There are four snap tabs, two on each side of the Wire Cover. Using a small, slotted screwdriver, press in and lift up on two tabs along the same side of the Wire Cover. (fig. #2, #3 & #4)

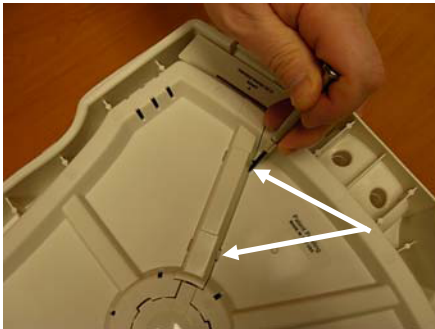


Figure #2

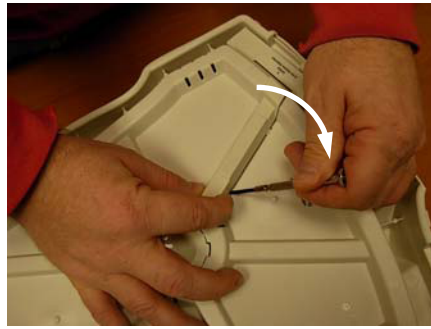


Figure #3

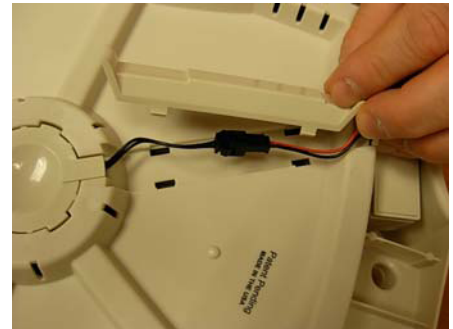


Figure #4

- **2) Disconnect the Wires** – Once the Wire Cover is removed, you will see wires from the Rate Monitor and from the Clicker leading to a black Connector in the middle. (fig.#5) Squeeze the small latch on the Monitor side of the Connector (fig.#6) and pull the two halves apart.(fig.#7)



Figure #5

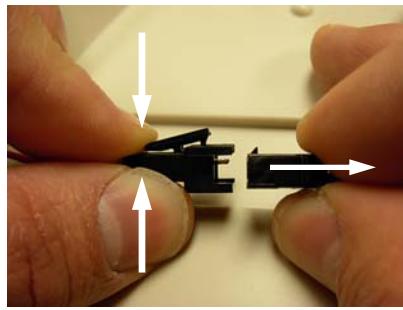


Figure #6

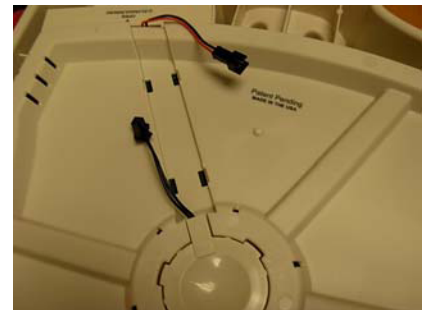


Figure #7

v.1-20-09

Replacing the 'Clicker' (Continued)

- **3) Remove the Old Clicker** - There are three tabs on the 'Clicker' that hold it in place. Using your screw driver, carefully press in and lift up on each tab while gently lifting up on the wire. (fig's #8 & #9) After the 3rd tab is unhooked, lift out the Clicker. Make sure the Spring is left down in the hole (fig.#10)

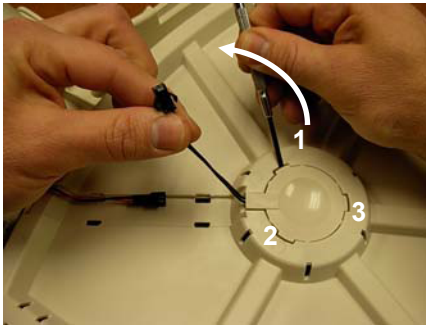


Figure #8

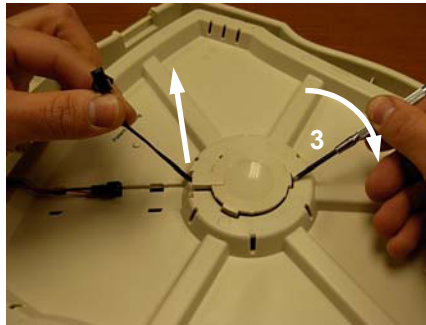


Figure #9

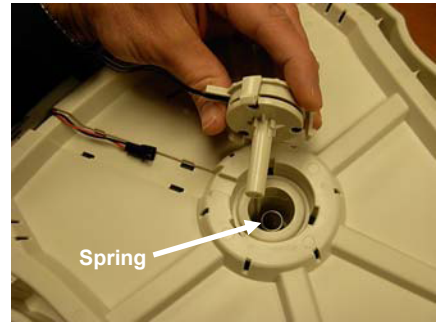


Figure #10

- **4) Insert the NEW Clicker** – Align the new Clicker with the hole, and snap it down into the hole, being careful not to pinch the wire. (fig. #11) Press firmly on each of the 3 tabs to insure they are engaged (fig.#12) Note: that New Clickers have a Black Dot in the center.

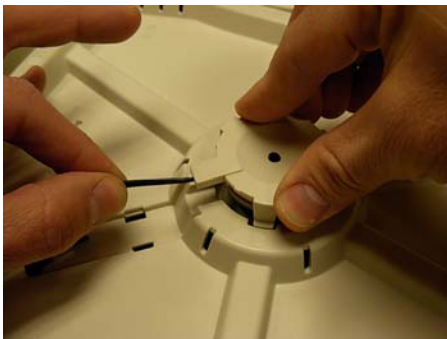


Figure #11

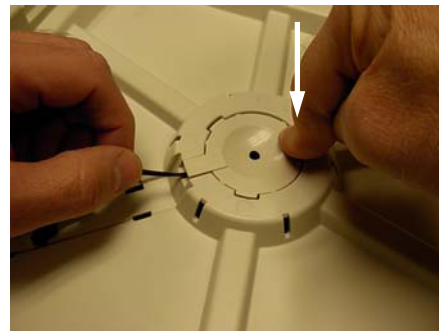


Figure #12

- **5) Re-connect the Wires and Replace the Wire Cover** – Align the Connectors and press them together. Make sure you hear a click. (fig.# 13) Align wires down the center of channel for the Wire Cover. If needed, make a loop with any excess wire. (fig.#14) Snap the Wire Cover into place by pressing firmly over 4 snaps (fig.#15)

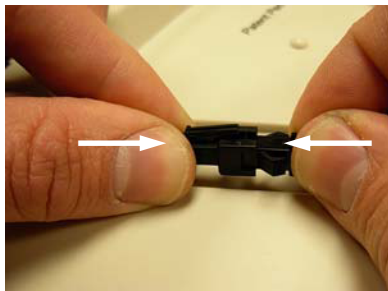


Figure #13

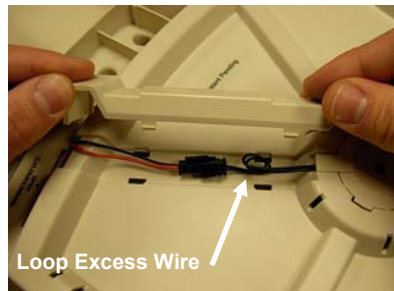


Figure #14

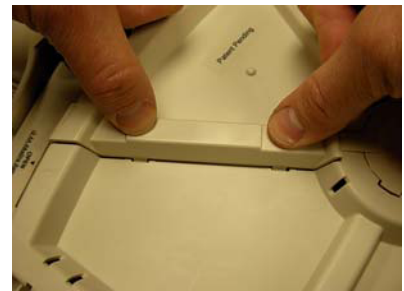


Figure #15